JOUPAD



Attention

The F7952W/J electrodes is only used on specified defibrillator models made by Jousing Medical.

Basic Information

REF	F7952W/J
Туре	Disposable, Self-Adhesive defibrillation pads, Latex free

Instructions for Use



Remove all clothing and expose the bare chest



Peel the electrodes from the protective sheeting



Tear open the package and take out the electrodes



Stick the electrodes onto correct position on patient

Operating and Storage Environment

Operating temperature: 0°C ~ 50°C

Recommended storage temperature: 5°C ~ 35°C

Extreme storage temperature: -30°C or 65°C, <24 hours

Relative humidity: 5% ~ 95%

Do not tear the pads package until you need to use the electrodes, Do not use dried out electrodes.

Use for the adult patients (> 25kg or 8 years old), Do not delay therapy to determine actual weight or age of patient during emergency.

The electrodes should be completely attached to the naked skin of the patient, Make sure the placed electrodes do not touch each other.

Symbols



















Made for Jousing Medical Co., Ltd

BLDG B2-303, Biobay, 218 Xinghu Street, Suzhou Industrial Park District, 215021

Suzhou, PEOPLE'S REPUBLIC OF CHINA.

Tel +86-512-62995391 Email service@jousing.com